Elos Accurate® Prosthetic Screwdriver

Instruction for use – Инструкция за
употреба - Upute za upotrebu - Návod k
použití – Brugsvejledning –
Gebrauchsanleitung – Οδηγίες χρήσης –
Instrucciones de uso – Kasutusjuhised –
Käyttöohjeet – Mode d'emploi – Használati
útmutató – Notkunarleiðbeiningar –
Istruzioni per l'uso – Naudojimo instrukcijos –
Lietošanas norādījumi – Bruksanvisning –
Gebruiksaanwijzing – Instrukcja użytkowania
– Instruções de uso – Instrucțiuni de utilizare
- Návod na použitie - navodila za uporabo -
Bruksanvisning - Kullanma Talimatı

Disclaimer of liability: Elos Medtech Pinol products may only be used according to the instructions and recommendations. The user of Elos Medtech Pinols products should determine their suitability for particular patients and indications. Elos Medtech Pinol disclaims any liability, expressed or implied, and shall have no responsibility for any direct, indirect, punitive or damages arising out of or in connection with any errors in professional judgement or practice in the use or placement of the Elos Medtech Pinol products.

Please note that not all products are regulatory cleared/released for sale in all markets even though the language is present in this instruction

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English - Instruction for use (English)

Elos Accurate® Prosthetic Screwdriver

1. Intended use

The Elos Accurate® Prosthetic Screwdriver is intended to be used by a dentist or dental technician for the fixation of hexalobular Process- and Prosthetic screws during dental procedures.

2. Product Description

The product consists of the Prosthetic Screwdriver which is manufactured from stainless steel. The product is available in different lengths. For specific product descriptions, please refer to individual product labels.

3. Indications

The Elos Accurate® Prosthetic Screwdriver is used for tightening or loosening of hexalobular Process- and Prosthetic screws from the Elos Accurate® assortment during dental procedures.

4. Contraindications

- The Elos Accurate® Prosthetic Screwdriver is not intended for tightening or loosening of Process or Prosthetic screws at an angle of more than 28° relative to the axis of the screw.
- The Elos Accurate® Prosthetic Screwdriver is not intended for tightening of screws with a torque above 35 Ncm.

5. Warnings and precautions

- Elos Accurate® Prosthetic Screwdriver must be secured against inhaling and swallowing during intraoral procedures.
- Do not use if damaged. Always inspect the product before use.
- Never clean product with metal brushes or steel wool.
- Elos Accurate® Prosthetic Screwdriver must only be handled by dental professionals.
- Always tighten the prosthetic restoration according to the implant manufactures instruction.
- Inspect Elos Accurate® Prosthetic Screwdriver under normal lighting for the removal of visible soil and for damage or wear: Any Elos Accurate® Prosthetic Screwdriver showing visible damage or wear must be disposed of according to section 12.

6. Compatibility information

The Elos Accurate® Prosthetic Screwdriver is operated manually by use of a handle or torque wrench which must be ordered separately.

7. Cleaning and Sterilization

The Elos Accurate® Prosthetic Screwdriver is delivered non-sterile. Prior to intraoral use it must be cleaned, disinfected and sterilized. Further instruction can be found in "Cleaning and sterilization guideline" at <https://elosmedtech.com/IFU/>.

1. Clean according to the guidance in the "Cleaning and sterilization guideline".

2. Sterilize according to the guidance in the "Cleaning and sterilization guideline". Summarized below.

Note: For US: Steam Sterilization Cycle 132°C (270°F)/4min.

Procedure	Dynamic-air-removal sterilization Cycle	
Exposure time	4 min.	3 min.
Temperature	132°C (270°F)	134°C (273°F)
Drying time	20 min.	

Caution:

Accompanying packaging material CANNOT be autoclaved unless it is explicitly stated that the material is suitable for sterilization.

8. Procedure

Gently open the blister box and take out the Prosthetic Screwdriver. Prior to intraoral use clean and sterilize according to the description above. Insert the Prosthetic Screwdriver in compatible handle or torque wrench supplied by Elos Medtech.

9. Further information

For additional information about the use of Elos Medtech products, please contact your local sales representative.

10. Validity

Upon publication of this instruction for use, all previous versions are superseded.

11. Storage and Handling

Recommended storage temperature range is 10-40°C (50-104°F) and the recommended relative humidity range is 0-80%

12. Disposal

When disposed of The Elos Accurate® Prosthetic Screwdriver may present an infection or microbial hazard and should therefore be disposed of as biological waste, according to national requirements.

13. Serious Incidents:

If any serious incidents occur in relation with this device, they should be reported to Elos Medtech Pinol A/S and to the competent authorities in the country where the incident occurred. A serious incident is a situation where the device directly or indirectly led or might have led to death, a serious deterioration in the health of a person or a serious public health threat. Contact Elos Medtech Pinol A/S at: complaint.empi@elosmedtech.com

Caution: U.S. Federal Law restricts this device to sale by or on the order of a licensed dentist or physician.

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adresinden Elos Medtech Pinol A/S ile temasa geçin