Elos Accurate® Prosthetic Screwdriver

Instruction for use – Инструкция за
употреба - Upute za upotrebu - Návod k
použití – Brugsvejledning –
Gebrauchsanleitung – Οδηγίες χρήσης –
Instrucciones de uso – Kasutusjuhised –
Käyttöohjeet – Mode d'emploi – Használati
útmutató – Notkunarleiðbeiningar –
Istruzioni per l'uso – Naudojimo instrukcijos –
Lietošanas norādījumi – Bruksanvisning –
Gebruiksaanwijzing – Instrukcja użytkowania
– Instruções de uso – Instrucțiuni de utilizare
- Návod na použitie - navodila za uporabo -
Bruksanvisning - Kullanma Talimatı

List of Content

<i>English - Instruction for use (English)</i>	3
<i>български – Инструкция за употреба (Bulgarian)</i>	4
<i>Hrvatski – Upute za upotrebu (Croatian)</i>	5
<i>Čeština - Návod k použití (Czech)</i>	6
<i>Dansk – Brugsvejledning (Danish)</i>	7
<i>Deutsch – Gebrauchsanleitung (German)</i>	8
<i>Español - Instrucciones de uso (Spanish)</i>	10
<i>Eesti keel – Kasutusjuhised (Estonian)</i>	11
<i>Suomi – Käyttöohjeet (Finnish)</i>	12
<i>Français - Mode d'emploi (French)</i>	13
<i>Magyar - Használati útmutató (Hungarian)</i>	14
<i>Íslenska – Notkunarleiðbeiningar (Icelandic)</i>	15
<i>Italiano - Istruzioni per l'uso (Italian)</i>	16
<i>Lietuvių k. - Naudojimo instrukcijos (Lithuanian)</i>	17
<i>Latviski - Lietošanas norādījumi (Latvian)</i>	18
<i>Norsk – Bruksanvisning (Norwegian)</i>	19
<i>Nederlands – Gebruiksaanwijzing (Dutch)</i>	20
<i>Polski - Instrukcja użytkowania (Polish)</i>	21
<i>Português - Instruções de uso (Portuguese)</i>	22
<i>Română – Instrucțiuni de utilizare (Romanian)</i>	23
<i>Slovenský jazyk – Návod na použitie (Slovak)</i>	24
<i>Slovenščina – navodila za uporabo (Slovenian)</i>	25
<i>Svenska – Bruksanvisning (Swedish)</i>	26
<i>Türkçe – Kullanma Talimatı (Turkish)</i>	27

English - Instruction for use (English)

Elos Accurate® Prosthetic Screwdriver

1. Intended use

The Elos Accurate® Prosthetic Screwdriver is intended to be used by a dentist or dental technician for the fixation of hexalobular Process- and Prosthetic screws during dental procedures.

2. Product Description

The product consists of the Prosthetic Screwdriver which is manufactured from stainless steel. The product is available in different lengths. For specific product descriptions, please refer to individual product labels.

3. Indications

The Elos Accurate® Prosthetic Screwdriver is used for tightening or loosening of hexalobular Process- and Prosthetic screws from the Elos Accurate® assortment during dental procedures.

4. Contraindications

- The Elos Accurate® Prosthetic Screwdriver is not intended for tightening or loosening of Process or Prosthetic screws at an angle of more than 28° relative to the axis of the screw.
- The Elos Accurate® Prosthetic Screwdriver is not intended for tightening of screws with a torque above 35 Ncm.

5. Warnings and precautions

- Elos Accurate® Prosthetic Screwdriver must be secured against inhaling and swallowing during intraoral procedures.
- Do not use if damaged. Always inspect the product before use.
- Never clean product with metal brushes or steel wool.
- Elos Accurate® Prosthetic Screwdriver must only be handled by dental professionals.
- Always tighten the prosthetic restoration according to the implant manufactures instruction.

6. Compatibility information

The Elos Accurate® Prosthetic Screwdriver is operated manually by use of a handle or torque wrench which has to be ordered separately.

7. Cleaning

The Elos Accurate® Prosthetic Screwdriver is delivered non-sterile. Prior to intraoral use it must be cleaned, disinfected and sterilized. The following cleaning and disinfection method is recommended prior to sterilization:

- Clean rinsing under flowing water while brushing with an adequate brush.

8. Sterilization

The Elos Accurate® Prosthetic Screwdriver is delivered non-sterile. The recommended sterilization procedure is full cycle pre-vacuum steam sterilization at a temperature of 134 °C (273°F) for 3 minutes, 3 pulse minimum.

Caution: The device must be used immediately after sterilization. Do not store sterilized devices.

9. Procedure

Gently open the blister box and take out the Prosthetic Screwdriver. Prior to intraoral use clean and sterilize according to the description above. Insert the Prosthetic Screwdriver in compatible handle or torque wrench supplied by Elos Medtech.

10. Further information

For additional information about the use of Elos Medtech products, please contact your local sales representative.

11. Validity

Upon publication of this instruction for use, all previous versions are superseded.

12. Storage and handling

Elos Accurate® Prosthetic Screwdriver should be stored at room temperature.

13. Disposal

The Elos Accurate® Prosthetic Screwdriver must be disposed as biological waste.

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Elos Medtech Pinol A/S